



Valley Contax • 200 South Mill St. • Springfield, OR 97477 • (800) 547-8815

Valley 15/16/18 Fitting Guide

Lens Fitting

1. Begin fitting 2.00D steeper than flat K.
2. With a diagnostic set, steepen lenses to achieve limbal vaulting without compromising health of scleral vessels.
3. A good fit is achieved with 150 microns of initial central clearance.
4. Evaluate periphery for vessel seal-off.
5. **After fit is validated, over refract and customize with a Valley Contax GP consultant.**